



**Connect**  
with ProSix.

Fore to hind, from the most distal tendon in the hind to the brain and all parts in between, ProSix is the neuromuscular connection that brings back "the signal", returning **strength, proprioception, and balance**. ProSix is made of knitted, ventilated elastic that provides support and gentle resistance to the horse's own movements. The tension in the elastic creates an external stimuli on stretch receptors in the muscles to provide information to the neurons and the brain, and that brings the injured or non-communicating area back "online".



# Prevent injury with ProSix.

ProSix builds **strength** through isometric and concentric exercise with the gentle resistance of the elastic framing the whole body.

ProSix connects the whole horse, stimulating **proprioception**, sometimes referred to as the sixth sense, creating whole body awareness and extending to all four limbs.

The ProSix surrounds with gentle deep touch pressure which can relieve stress and anxiety.

ProSix encourages correct posture with strategically placed support; improving **balance**, straightness, and rhythm, with more even weight distribution between all four legs.

## Use your ProSix:

- Proactively for creating balance, alignment, rhythm, strength, straightness, and calmness.
- To enhance length of stride, and engage hindquarters.
- To train in proper alignment and correct posture.
- To improve balanced, equal weight distribution between all four limbs.
- To build core strength, strong abdominal muscles, and create a strong, soft, supple back.
- As a diagnostic tool. ProSix highlights areas of injury by discouraging compensation.

# Rehabilitate with ProSix.

With ProSix, rehabilitation can start earlier, even immediately post-operatively in many cases. The horse receives the benefits of calming, strengthening, increased proprioceptive awareness, balanced weight bearing, and supportive deep touch pressure by simply wearing this comfortable, non-restrictive wrap. ProSix is ideal for the horse with activity restrictions, from stall rest to hand-walking, to supervised limited turnout.

## ProSix is a broad-spectrum rehabilitative tool for:

- Rehabilitation for neurological problems following veterinary diagnosis and treatment. (ex: parasites, illness or injury resulting in neurological deficits such as EPM, Wobblers, EHV-1, or West Nile Virus).
- Maintaining weight distribution among all four limbs, bilaterally and fore to hind, to prevent overload to non-injured limbs.
- Encouraging correct posture and balance. May also help to maintain chiropractic adjustments.
- Use following colic or other abdominal surgeries to provide support and strengthen abdominal muscles.
- Use with back pain and spinal issues to provide support with strengthening. Effective for treating impingement of dorsal spinous processes, muscle strain, other causes of back pain.
- Calming and reducing anxiety with deep touch pressure and gentle resistance band exercise. This makes handling during rehabilitation and treatment safer for any handler or equine professional.

## Rehabilitation made easy with ProSix.

**EASY** to observe improvement with lasting results.

**EASY** to put on and take off.

**EASY** for the horse to wear.

**EASY** to adjust.

**EASY** to clean. Hand wash, hang dry.

**ProSix is easy for clients to use.** It is comfortable, supportive, and non-restrictive for the horse allowing normal range of motion. It can be worn during hand-walking and stall rest when activity needs to be restricted. It is safe to be worn while lunging, in round pen work, on a walker, or at liberty in a small turnout.

ProSix can also be worn under or over any type of saddle, or with harness. There are no rigid treatment protocols; the horse should be observed for fatigue, exercise intolerance, or any increase in pain or discomfort. In a small, safe turnout, most horses will limit activity as needed.

Veterinarians working with owners create the optimum protocol with adjustments made as the horse progresses in its treatment. We are available for consult based on our experiences with other cases.

**ProSix is available in five sizes to provide optimal fit for your horse.** See size chart on website.

A single ProSix may be used on multiple horses.

## Ines Wecker | Veterinarian and Chiropractor

Director of the Backbone-Academy, Buxtehude, Germany

“

I am a vet of over 25 years and an animal chiropractor of over 15 years. I must say, ProSix is outstanding for sensory as well as motor neuromuscular rehabilitation. Without ProSix, some of my tough patients wouldn't have made it.

But that shouldn't imply that it is only useful for last hope cases—the opposite. In the best of all worlds, we and our horses would work in perfect balance all the time. As we're all far from that, our horses actually do gain so much from the ProSix. Get in contact with ProSix and see for yourself!

”



MADE IN THE U.S.A.

[info@eagleprosix.com](mailto:info@eagleprosix.com)

Jill (906) 241-8540

[www.eagleprosix.com](http://www.eagleprosix.com)

Lynda (906) 398-8237

The material found on our website and in other promotional materials is provided for educational purposes only and is not to be used for medical advice, diagnosis or treatment. Please seek veterinary advice prior to use as a treatment for rehabilitation or for reconditioning. Neither Eagle Link, LLC nor any authorized retailer of Eagle ProSix will be held liable for any personal accident, injury or damage resulting from the use of the Eagle ProSix. Full responsibility lies with the individual who chooses to use ProSix or ProSix K9 for their horse or dog under his or her care.